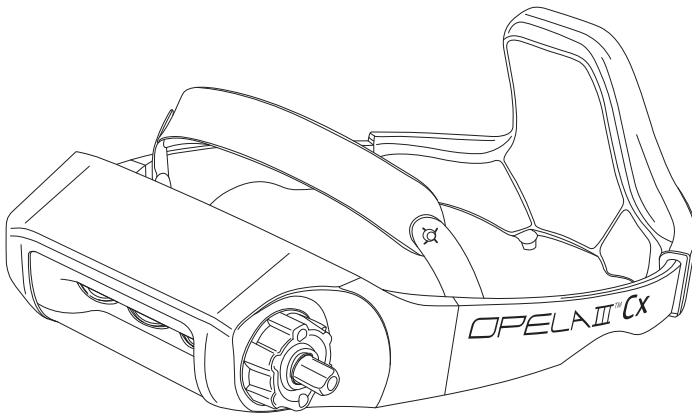
Wearable OR Lighting System

# OPELA III™ Cx

Safety Precautions

Components

Instruction for Use

Troubleshooting

Specifications

Warranty

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Thank you for purchasing O P E L A III® Cx.

Please be sure to read this instruction manual before use.  
Please also keep this manual close at hand for future reference.

**1****Safety Precautions**

Please read the instruction for use carefully before using the product.

**[Intended use]** This product is intended to illuminate the view field during surgery, examination, and treatment. Please make sure that the product is only used for the above purpose. It is designed to be worn on the head.

**[Intended users]** This product is intended to be used by a surgeon.

**[Intended patients]** Children and Adults

**[Place of use]** Inside medical facilities

The safety precautions are classified into and clearly specified as "Warning" and "Caution". Be sure to follow them during use.

 <b>WARNING</b>	Indicates potential death or serious injury if this description is ignored and improperly handled.
 <b>CAUTION</b>	Indicates potential mild or moderate injury if this description is ignored and improperly handled.
<b>CAUTION</b>	Indicates expected failure, damage, malfunction, or defects if this description is ignored and improperly handled.

- [ ⚠ WARNING ]**
  - Explosion may occur. Do not use in a flammable gas atmosphere.
  - Malfunction or accident may occur if the product is dropped. Please do not use this product for any other purpose other than wearing on your head.
  - Prolonged use at high intensity or short distances may cause burns. Please do not use at high intensity or short distances for long period.
- [ ⚠ CAUTION ]**
  - Hazardous optical radiation may be emitted from this product. Do not stare at operating lamp or illuminate the eyes for too long. May be harmful to the eyes.
  - It may cause burns. Do not use for photosensitive patients or patients who have received photosensitizing agents.
  - Electric shock may occur, if the device is damaged. Discontinue use immediately in such case and do not damage the cables.
  - Electric shock may occur. Do not touch the patient and the charging stand plugged into power supply via AC adaptor at the same time.

**[In case of a serious accident]**

In the event of any serious incident occurring in connection with the device, it must be notified to the manufacturer and the relevant authority of the member state in which the user and/or patient is based.

**■ Table of symbols**

Symbol	Description
	Caution
	Refer to instruction manual/booklet
	Manufacturer
	Date of manufacture
	Country of manufacture
	Serial number
	Waste electrical and electronic equipment
	Direct current

Symbol	Description
	Rated power input
	Rated power output
	Rated power input (battery)
	Plug polarity
	Medical Device
	Authorized representative in the European Community
	Importer

## 2

# Components

This product consists of the following items.

### ■ Components

Name	Quantity
Main unit (with COOLMAX® inner pad)	1 pc
Battery unit	2 pc
Charging stand	1 pc
AC adaptor	1 pc
Attached belt (LL)	1 pc
Storage case	1 pc
Shoulder belt for storage case	1 pc
Instruction for use	1 copy

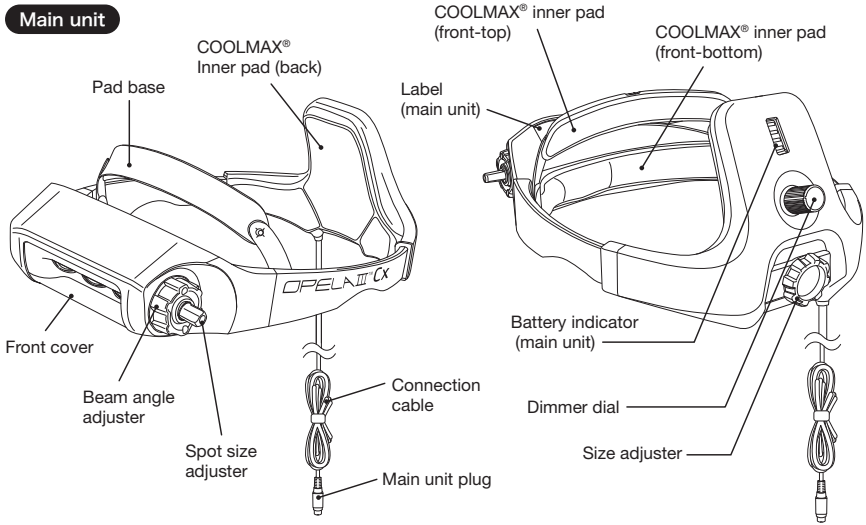
**COOLMAX**

This product uses "COOLMAX®" as interior fabric which allows sweat to evaporate for lasting comfort.


- COOLMAX® exhibits superior moisture control by allowing perspiration to escape, while letting air enter, keeping the fabric cool and dry, effectively lowering surface temperature.
- COOLMAX® is soft, lightweight with natural texture and excellent air permeability.

※ COOLMAX® is a registered trademark of INVISTA.


**Main unit**


**Battery unit**

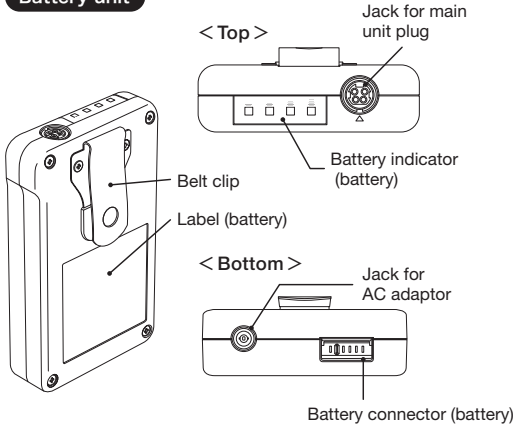**AC adaptor**

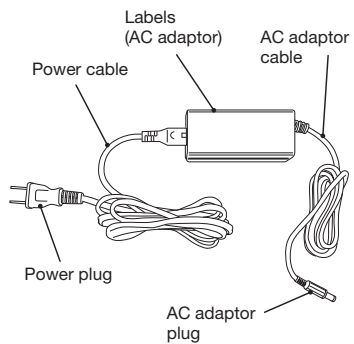**Charging stand**
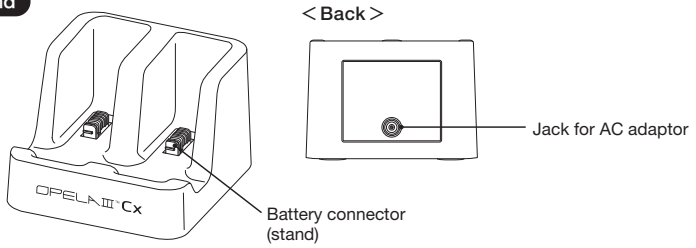
# 3

## Instruction for Use

Before use

Wear and adjust

Use

After use

### 3-1

## Charging the battery

### ⚠ CAUTION

Electric shock may occur.  
Do not damage the cables.

Electric shock may occur.  
Do not touch the patient and the charging stand plugged into power supply via AC adaptor at the same time.

### CAUTION

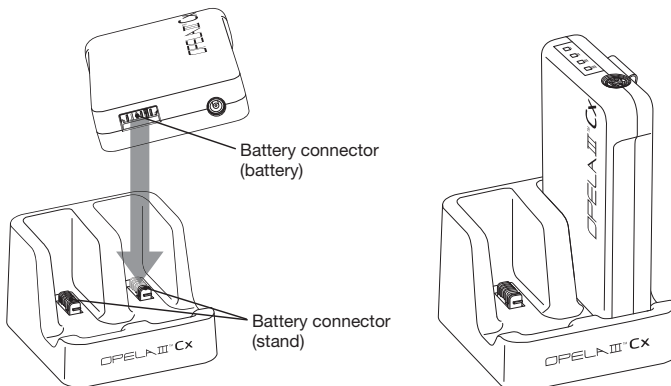
Malfunction may occur.  
Do not use any accessories which are not included in this product.

1. Place the battery unit on the charging stand. Please make sure that the battery connector on both sides click into place.

\*Connecting the AC adaptor directly to the battery unit will not charge the battery. Please use a charging stand to charge the battery.


\*When using the AC adaptor, keep it in a state where it can be disconnected from the power source at any time in case of unforeseen event.

\*When using the battery for the first time or if it has not been used for more than 3 months, be sure to recharge it before use. At that time, charging may end before the battery is fully charged. In this case, please repeat charging and discharging 2 to 3 times before use.

2. Make sure the battery indicator (battery) is flashing green.

**Battery indicator**

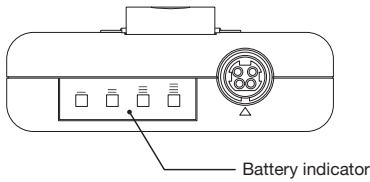
- : Green light on
- (with radiating lines) : Green flashing
- (grey) : Orange light on
- : Lights off


0-25% charging

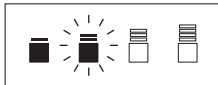25-50% charging

50-75% charging

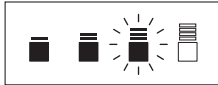75-99% charging

Fully charged

**Error**

Error

No response

If it shows error or does not respond, remove the battery from the stand and insert it again until it clicks into place. If it still does not flash green, please refer to Troubleshooting.

## 3-2 Pre-use inspection

### ⚠ CAUTION

Hazardous optical radiation may be emitted from this product.  
Do not stare at operating lamp or illuminate the eyes for too long. May be harmful to the eyes.

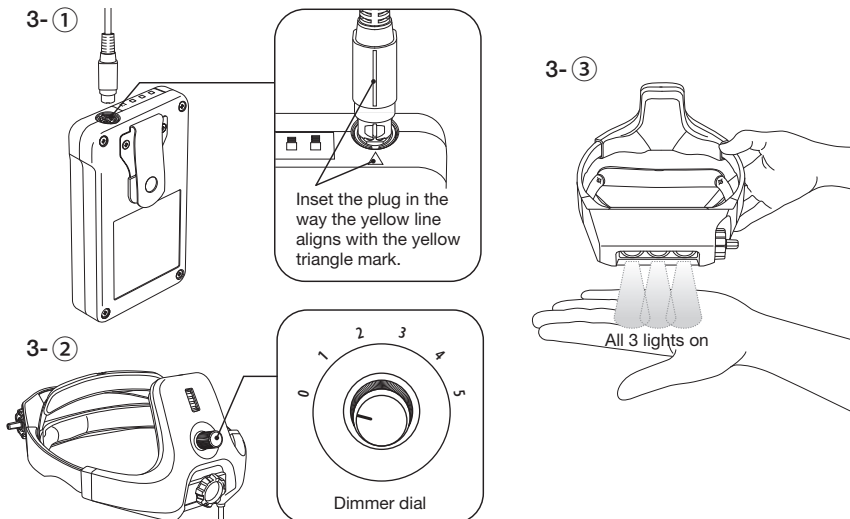
### CAUTION

Malfunction may occur.  
Do not insert anything other than the enclosed plugs into the jack.

1. Shake the main unit, battery unit, and AC adaptor to make sure there are no abnormal sounds.
2. Turn all dials and make sure they are not loose.
3. Connect the main unit to the battery, turn the dimmer dial on the rear of the main unit, make sure all three LEDs are turned on at all levels.
4. Maximum illumination may not be obtained if the front cover is covered with fingerprints or other smudges. Check the front cover for smudges, and if there are any, wipe them off with ethanol before use.

\*Do not look directly into the LEDs, but hold the lights so that it projects the light against your hand.

\*If there is an abnormal sound, or if any dials are loose, or if any of three lights is not lit, stop using the product and contact the dealer.

### 3-3 Wearing (Attached belt, battery unit)

#### ⚠ CAUTION

Do not use with other belts.

This may cause the attached belt to loosen and the battery unit to fall. (Be sure to use the attached belt of this product.)

Wear the attached belt for a couple of hours before use.

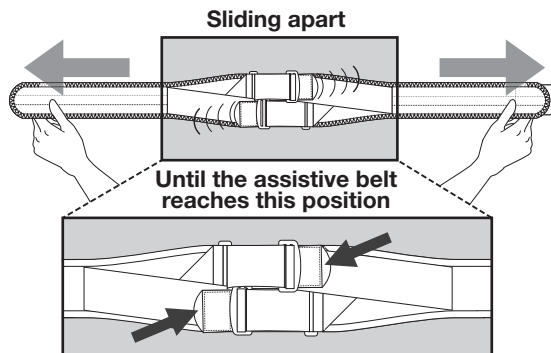
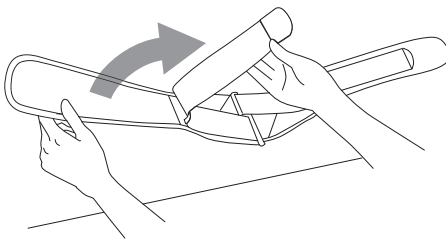
Wear it for a couple of hours at first to check that there is no discomfort or itchiness. You can use it all day once it fits comfortably.

Wrap the attached belt around your pelvis so that it does not loosen.

The battery unit may fall if the attached belt loosens or moves around.

Make sure to put on the attached belt by following the steps in the correct order.


1. Remove the assistive belt and pull the main belt sideways in both directions.


**2.** Place the attached belt on your pelvis.

\*Wear the attached belt underneath your surgery gown.

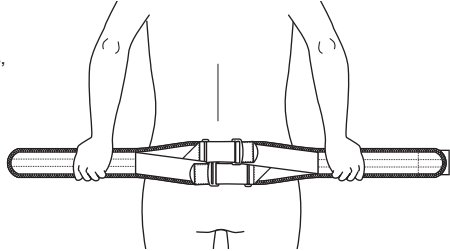
\*Wear the attached belt around your pelvis, not your waist.

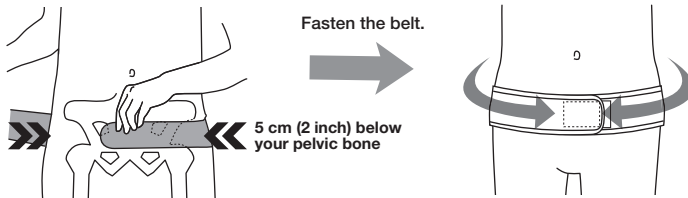
**3.** Wrap the attached belt around your pelvis.

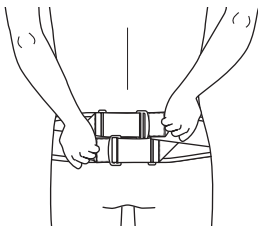
**4.** Hold the assistive belt at the back.


**6.** Fasten the assistive belt.

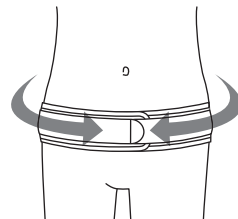**5.** Pull the assistive belt.

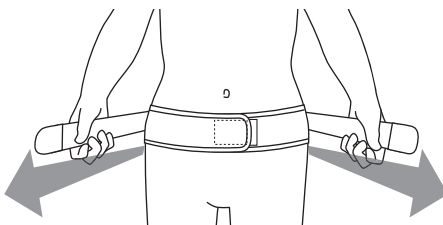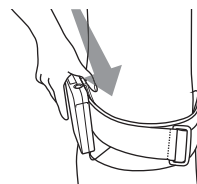
**7.** Insert the belt clip of the battery unit deeply and tightly on the backside of the main belt.


## 3-4 Wearing the battery unit

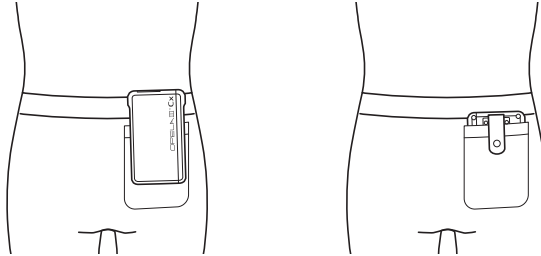Place the belt clip of the battery unit in the waistband of your pants or in the back pocket of your pants.

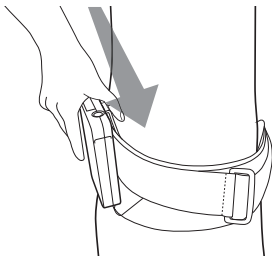The belt supplied with OPELAIII can also be used.


## 3-5 Wearing the main unit

### ⚠ WARNING


Malfunction or accident may occur if the product is dropped.  
Please do not use this product for any other purpose other than wearing on your head.

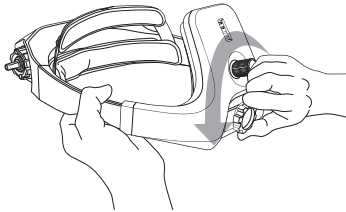1. Turn the adjuster dial to the left to increase the size.


2. Put it over your head and turn the adjuster dial to the right to tighten it.

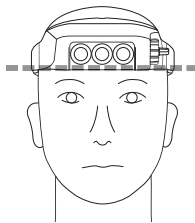\*Tighten it up until it does not fall off even if you shake your head.



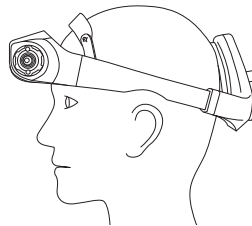
Nearly level  
Go a little over the eyebrows

### Using with loupe

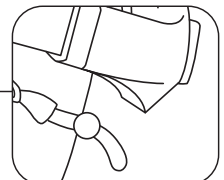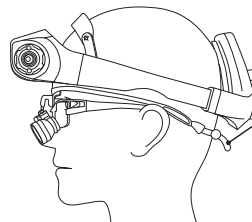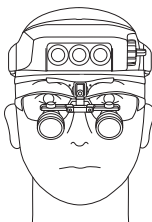To prevent the OPELAIII Cx from interfering with the loupe, fix the front of the OPELAIII Cx.

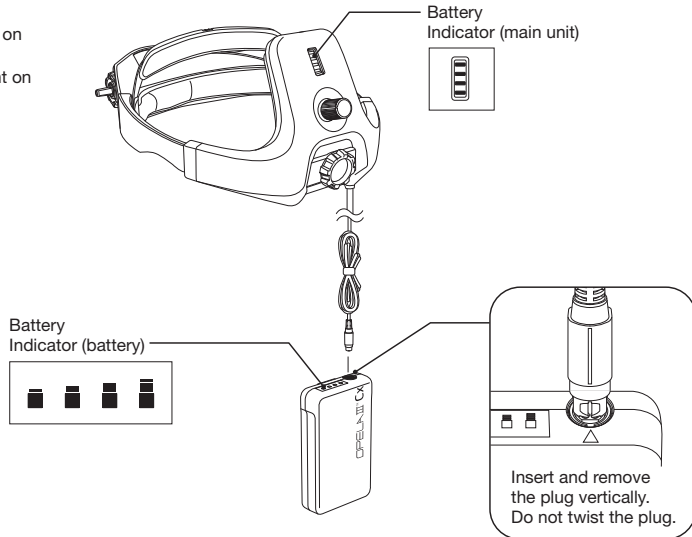
Careful not to allow the loupe clasp, gathered hair, etc. to get between the product and the head.

## 3-6 Supplying power to the main unit

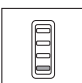
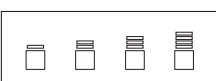
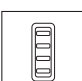
There are two ways to supply power to the main unit.

### 1. How to supply power from the charged battery.

-  : Green light on
-  : Orange light on
-  : Lights off

#### Power left

75-100%		
50-75%		
25-50%		
10-25%		
0-10%		
0%		

#### Error

##### Error

##### No response

If it shows error or does not respond, remove the battery from the stand and insert it again until it clicks into place. If it still does not flash green, please refer to Troubleshooting.

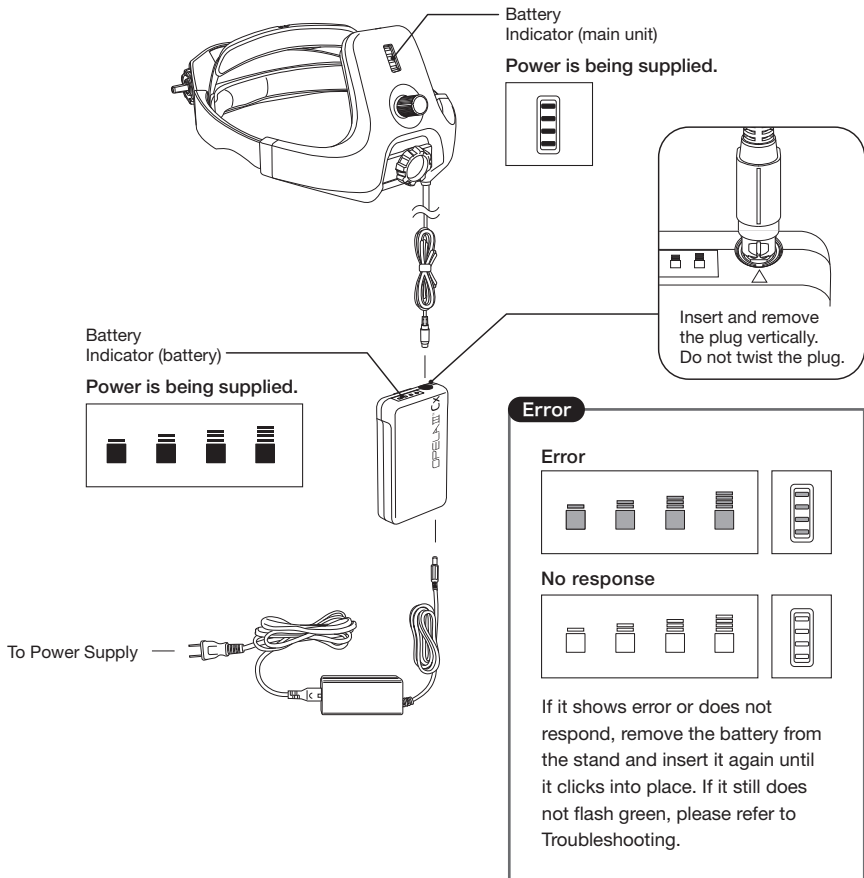
## 2. How to supply power from the power outlet via the AC adaptor and the battery while in use.

All four lights are on green while the power is supplied from the outlet. No change in the indicator is made.

\*Battery is not charged while the power is supplied to the main unit from the outlet. Please charge the battery using the charging stand.

\*When using the AC adaptor, keep it in a state where it can be disconnected from the power source at any time in case of unforeseen event.


-  : Green light on
-  : Orange light on
-  : Lights off

## 3-7 Adjusting the light

### Adjusting the angle

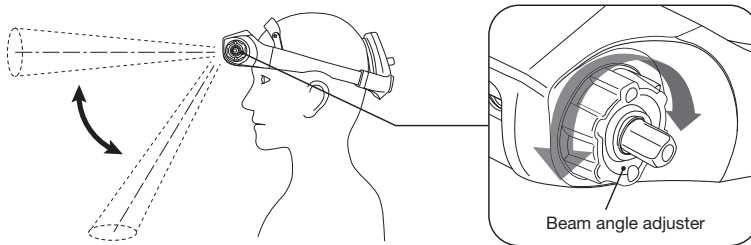Turn the beam angle adjuster forward to illuminate the lower field, and turn it back to illuminate up front.


### Adjusting the spot size

Turn the spot size adjuster forward to make the spot bigger, and turn it back to make it smaller.


\*Clicking sound will be heard when the adjuster reaches its limit.

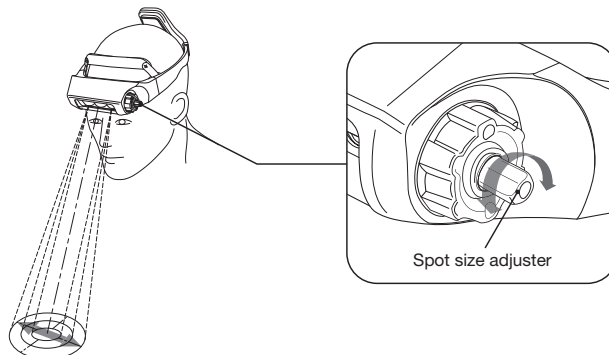\*When the beam angle adjuster is facing straight down, the spot size adjuster tends to spin without change to the size, so turn the beam angle adjuster slightly upwards and then adjust the size again.

### Brightness level adjustment

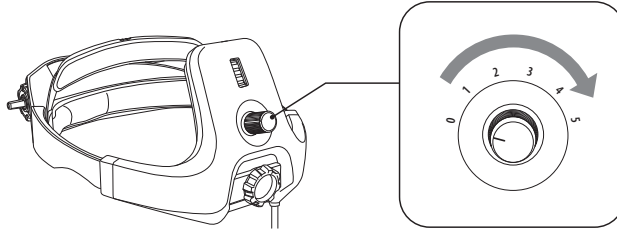Turn the dimmer dial to the right to turn on the light and adjust the illuminance.

Dimming level	Illuminance	Lighting time
5	160,000 Lux	4 hrs
4	135,000 Lux	5 hrs 30 min
3	100,000 Lux	7 hrs 30 min
2	80,000 Lux	12 hrs
1	40,000 Lux	20 hrs

\*The above is only a guideline for a brand new product.

\*Please adjust the dials by hand. No auto adjuster is mounted.

\*OPEL III Cx is not intended for sterilization. Please ask the staff around you to adjust the dials as necessary to prevent the dials from contaminating your hands.

## 3-8 Use

### ⚠ WARNING

Explosion may occur.  
Do not use in a flammable gas atmosphere.

Prolonged use at high intensity or short distances may cause burns. Please do not use at high intensity or short distances for long period.

### ⚠ CAUTION

Hazardous optical radiation may be emitted from this product. Do not stare at operating lamp or illuminate the eyes for too long. May be harmful to the eyes.

It may cause burns. Do not use for photosensitive patients or patients who have received photosensitizing agents.

Electric shock may occur, if the device is damaged. Discontinue use immediately in such case and do not damage the cables.

### CAUTION

Malfunction may occur.  
Use the product where it meets the environmental requirement.  
Do not bring the product into an MRI.

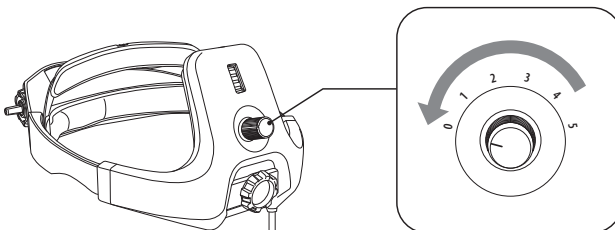
\*When this product is the only source of lighting during surgery, examination, or treatment, please ensure that backup lighting is also prepared.

### Stopping operation

Turn the dimmer dial to the level 0 to turn off the light.

\*When powered from electric outlet via AC adaptor, unplug the AC adaptor from the outlet.

\*In an emergency, the light can be turned off by unplugging the main unit cable from the battery unit.

## 3-9 Maintenance

### CAUTION

Malfunction may occur.

Do not disassemble, repair, or customize the product.

Do not use disinfectant other than ethanol.

Do not store the product where it cannot meet the environmental requirement.

Do not store the product in direct sunlight.

### 【Cleaning】

#### Main unit and battery unit

Wipe the product with ethanol after use.

#### COOLMAX® inner pad

Each inner pad can be removed for washing as needed.

- Dissolve a very small amount of pad shampoo or neutral household detergent in water or lukewarm water (35°C or less) and gently rub the pad with your hands, then rinse well.
- After washing, remove water gently with a dry cloth and dry in the shade in a well-ventilated place out of direct sunlight.

#### Storage case

Wipe with a wet cloth with a small amount of neutral detergent.

## 【Consumables】

Replace consumable parts as needed.

Consumable parts	Approximate timing of replacement
Battery unit	The estimated replacement time per unit is about 2 years. We recommend replacing battery units every two years, as they may suddenly stop working.
COOLMAX® inner pad	When deterioration or soiling is a concern.

## 【Maintenance】

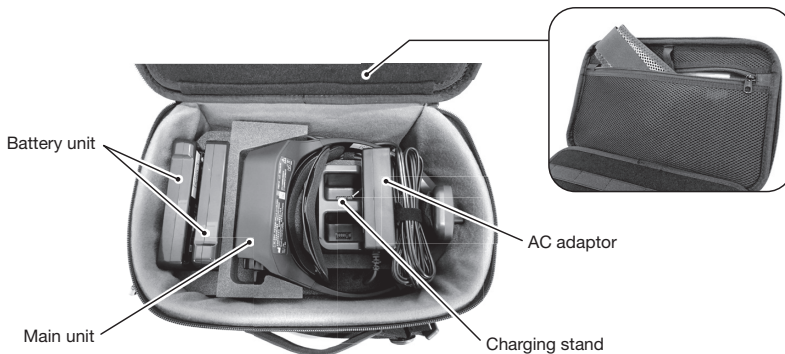
It is recommended to return the product to the manufacturer for the regular maintenance. Please note the maintenance cost and the shipping cost will be charged to you.

## 【Storage】

1. Turn the adjuster dial on the main unit to the right to minimize the size.
2. Store each component in a storage case as shown below.

\*It must be the smallest size or it will not fit in the storage case.

\*If you are using the attached belt, please place it in the pocket on the lid.

## 3-10 Disposal

When disposing of the product, please follow the regulations of each country. For details, please contact your local WEEE.

## 4

## Troubleshooting

Trouble area	Possible cause	Corrective action
The light does not turn on.	Dimmer dial is set to 0.	Turn the dimmer dial to the right.
	Insufficient remaining power in battery unit.	Charge the battery.
The light goes on and off.	Insufficient remaining power in battery unit.	Charge the battery.
The spot size adjuster spins without change to the spot size.	The beam angle adjuster is pointing straight down	Turn the beam angle adjuster slightly upwards and adjust the size again.
The light runs less than two hours even when fully charged.	Poor connection cord contact.	Remove any dirt from the plug-jack and insert and remove the plug from the jack 2 to 3 times.
	Battery unit is deteriorated.	Repeat charging and discharging 2 to 3 times.
Battery unit or AC adaptor gets hot during charging.	Due to the energy-exchanging heat during charging.	It is normal for the temperature to reach around 55°C.
		Charge the battery under the condition specified in "environment". (15-25°C).
Indicator does not respond even when the AC adaptor is connected.	Poor power plug contact.	Remove any dirt from the power plug and unplug it from the outlet 2 to 3 times. If there is still no response, try another outlet.
	Poor jack contact.	Remove any dirt from the jack and the plug of the AC adaptor.
All four lights of the indicator are lit orange.	The battery unit is reconnected immediately (within 1 second) after it was disconnected from the charging stand.	Disconnect the battery unit from the charging stand, wait about 1 second, and then reconnect it to the stand. The battery starts charging and the indicator starts blinking green.
	Only the battery unit and AC adaptor are connected.	Connect the main unit. All four lights of the indicator will be turned green.
	The temperature inside the battery has exceeded 60°C during charging.	Charging will resume automatically once the internal battery temperature drops to approximately 45°C. Please leave the battery unit on the charging stand.

\*In case the trouble is not resolved with above actions, please contact your local dealer as the device could be malfunctioned.

### ■ Classification

Electrical shock protection classification	Internally powered equipment (when using battery), Class II equipment (When powered from electric outlet via AC adaptor, Class II AC adaptor as a part of ME equipment)
Protection against harmful intrusion of water or particulates	Not applicable
Sterilization	Not applicable
Use in high oxygen concentration environments	Not applicable

Brand name	OPELAIII Cx
Illuminance at center (illumination range: ø60mm, illumination distance: 400mm)	160,000 Lux ± 10%.
Color temperature	Approx. 5,000 K
Illumination size (at 400mm illumination distance)	Approx. 60-230 mm dia.
LED	3 high-intensity LEDs (average life time 50,000 h)
Power consumption	Approx. 8W

### ■ Main unit

Rated input (when powered from a charged battery)	6V, 8W
Rated input (When powered from electric outlet via AC adaptor)	12V, 8W
External dimensions	Approx. W200 x H115 x D245-300 mm
Weight	Approx. 310 g
Head measurement	51-65 cm

### ■ Battery unit

Built-in rechargeable battery	Nickel hydride battery
Rated input (when power is supplied from a charging stand)	8V, 1.8A
Rated input (When powered from electric outlet via AC adaptor)	12V, 0.7A
Rated output (when using charged battery)	5V, 1.8A
Rated output (When powered from electric outlet via AC adaptor)	12V, 0.8A
External dimensions	Approx. W85 x H140 x D40 mm
Weight	Approx. 430 g

### ■ Charging stand

Rated input	12V, 2.5A
Rated output	8V, 3.6A
External dimensions	Approx. W100 x H65 x D100 mm
Weight	Approx. 250 g

### ■ AC adaptor

Rated input	100-240V, 47-63Hz, 0.4-0.6A
Rated output	12V, 2.5A

## ■ Environment

During charging	15-25°C, 20-85% RH (non-condensing)
During operation	0-35°C, 20-85% RH (non-condensing)
During storage / transport	20-50°C, 20-85% RH (non-condensing)
Atmospheric pressure allowance	700 to 1060 hPa

\*Specifications change depending on the usage conditions and are subject to change without notice.

\*The expected life time is 5 years on the condition that the maintenance by the manufacturer every 2 years.

\*Using, charging, or storing the product at a temperature or humidity outside of environmental specifications will reduce the battery capacity and charging efficiency and cause the lighting time to become shorter.

\*Photobiological safety (IEC62471)

This product is classified as Risk Group 2.

## **[Applied Standard]**

IEC60601-1

IEC60601-1-2

## **[Electromagnetic compatibility]**

If this product needs to be used in close proximity to other equipment, make sure that this product and other equipment are operating properly before use.

## **[Medical Device Regulation]**

This product complies with the requirements of European Regulation (EU) 2017/745.

Classification : Class I

## ■ Warranty coverage

- We will repair or replace the product free of charge if it fails within 1 year from the date of purchase provided it has been used correctly following the instruction for use and precaution notes of attached labels, etc.

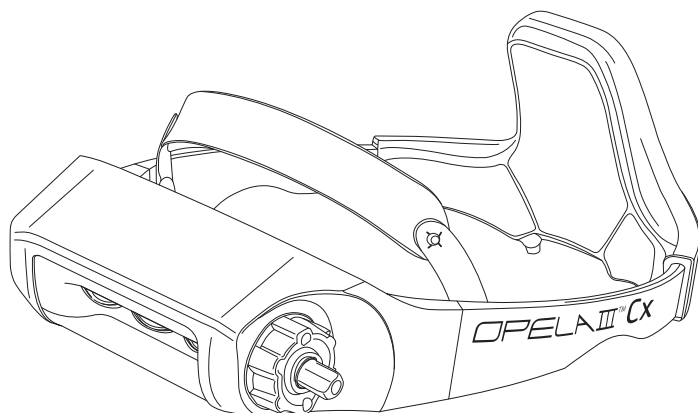
## ■ Disclaimer

- The warranty does not apply if the failure or malfunction falls under any of the following categories.
  - a) Defect or damage caused by incorrect use and/or inappropriate repair and modifications.
  - b) Defect or damage caused by dropping after purchase.
  - c) Defect or damage caused by fire, earthquake, flood, lightning, other natural disasters, pollution or abnormal voltage from the power source, power supply that is outside the specifications (voltage, frequency), etc.
  - d) Consumable parts. (battery unit, COOLMAX® comfort inner pads, cord clip, etc.)
  - e) Cause of defect is attributable to anything other than this product.
  - f) Any other defect and/or damage caused by usage not specified in this instruction manual.
  - g) Only the main unit, AC adaptor, and charging stand are covered.
- We shall not be liable for any incidental damages (loss of business profits, business interruption) arising out of the use or inability to use this product.

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**OPELA III**® is registered in U.S. Patent and Trademark Office.
**OPELA III**™ is trademark of Taiyo corporation.
**OZG OneZurich Group AG**
**MedNet EC-REP GmbH**